

# Inactivir™ Viral Inactivation Buffers DNA Sample Stability

## Inactivir™ Viral Inactivation Buffers stabilise DNA for sampling

#### **Purpose**

For the purposes of home sampling or even within near POC testing, the stability of the molecular target for analysis is critical to assay sensitivity. This may be necessitated by the site of collection, where the ability to analyse samples immediately may be limited<sup>1</sup>.


This application note quantifies the stability of a DNA target (a DNA version of portions of the SARS nCoV2 virus) in Viral Inactivation Buffers VIB-003 Inactivir™ SD LITE and VIB-004 Inactivir™ SD COMPLETE (referred to as VIBs in this note) over a 72 hour time-period with samples stored at room temperature (21 °C). This covers a time period up to that which could be expected for home sampling and postal return of samples. Sample stability is tested using pseudo-samples containing known quantities of bacteria and human cells to represent the major components expected in swab samples.

The SD Inactivir™ Viral Inactivation Buffers tested are chemically classified as non-hazardous. They have also been tested by PHE and shown to inactivate the SARS nCoV2 virus to the limit of detection of their assays.

The buffers are compatible with standard commercial RNA extraction kits. They have been validated against exemplars of the Qiagen™ QIAamp™ Viral RNA Mini Kit for silica spin columns and the Applied Biosystems™ MagMAX™ Viral RNA isolation kit for magnetic-bead based extractions. We have used an in-house purification method using EZ-10™ columns for the data presented in this report.

#### **Results**

Extractions were completed from 3 sets of duplicate samples. Each sample had the extracted DNA quantified in duplicates using QPCR detection with CDC N2 primers. The Ct values determined for each extraction are shown in Figure 1, with averages for the extractions shown by the red bars.

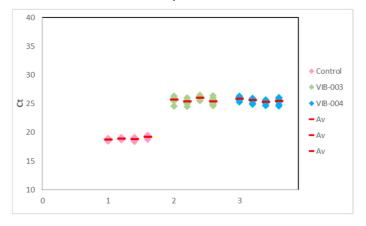

**Fig1** Ct values of samples stored at 21°C. Samples are 1: Viral DNA control, 2: VIB-003 Inactivir™ SD LITE, 3: VIB-004 Inactivir™ SD COMPLETE. Data for each are shown at 0, 24, 48 and 120 hours (left to right).

<sup>&</sup>lt;sup>1</sup> Blow et al Journal of virological methods, (2008) <a href="https://doi.org/10.1016/j.jviromet.2008.02.003">https://doi.org/10.1016/j.jviromet.2008.02.003</a>



# Inactivir™ Viral Inactivation Buffers DNA Sample Stability

These figures show the DNA control replicates have a Ct of around 19 and shows less than 1 Ct run to run variation. The extracted RNA from the pseudo samples (Fig 1) is in the range of Ct 25-26. This 6 Ct loss is primarily due to the 10-fold dilution caused by using 5  $\mu$ l in assays from 50 ul eluted from the purification. There is no significant variation in the Ct values from the recovered DNA over the 72 hour period. This shows no degradation of the samples across the 72-hour storage period.

#### **Conclusions**

The data shows excellent preservation of DNA within a pseudo sample made with a reference viral DNA target across a 72-hour time course at 21 °C. This demonstrates that these Inactivir™ Viral Inactivation Buffers are highly effective at preserving DNA in samples for subsequent analysis.

#### **Methods**

Pseudo-samples were made to replicate the environment of typical sampled viral DNA. These contained both human and bacterial cells, as would be expected in a swab or saliva sample, along with a DNA reference standard made from a plasmid containing SARS nCoV-2 sequence. This is a synthetic DNA encoding parts of the Covid genome routinely used for detection using the published CDC N2 primer set that has been extensively used for viral RNA detection. This allowed DNA stability to be assessed using the same procedures as have been widely established and validated previously. Samples were made in duplicate of 25  $\mu$ l of our DNA reference material plus 50  $\mu$ l of Hela cells (~10<sup>6</sup> ml<sup>-1</sup>) in Phosphate Buffered Saline (PBS) and 20  $\mu$ l of E. coli (~1 OD stock). To this sample 1 ml of the VIB under test was added.

Samples were purified using a 'homemade' kit based on a published extraction procedure outlined by the Baker lab and used by Addenbrookes Hospital, Cambridge, Uk². This has previously been validated to produce similar results to extractions from the commercial Qiagen™ QIAamp™ Viral RNA Mini Kit and the Applied Biosystems™ MagMAX™ Viral RNA isolation kit.

Samples were taken immediately after inactivation, and subsequently at each timepoint after storage at 21°C. From the stock 200  $\mu$ l samples were taken and extracted with the previously validated 'home made' protocol, as follows. 200  $\mu$ l of VIB inactivated sample was mixed with 100  $\mu$ l of absolute ethanol. This was added to the EZ-10 column and spun at 8k rpm for 90 sec. The flow through was discarded and 350  $\mu$ l of 3M Sodium Acetate (pH 5.2) wash buffer was added and spun again with the flow through discarded. The column was washed twice with 500  $\mu$ l 70% ethanol, with spun and discard. The column was then spun to dry at 12.5 k rpm for 2 min. Finally, the column was placed in a fresh tube, 50  $\mu$ l of ultrapure water was added and after 2 min the sample was spun and the purified DNA collected.

For detection of the DNA the CDC published N2 primer / probe combination<sup>3</sup> was used with a FAM / BHQ-1 quencher combination for the probe (Sigma Aldrich). Samples were amplified using PCR Biosystem qPCRBIO

<sup>&</sup>lt;sup>3</sup> https://www.cdc.gov/coronavirus/2019-ncov/lab/rt-pcr-panel-primer-probes.html



CliniSciences Group

<sup>&</sup>lt;sup>2</sup> bioRxiv preprint https://doi.org/10.1101/2020.04.14.041319



# Inactivir™ Viral Inactivation Buffers DNA Sample Stability

Probe 1-Step Go master mix. For each tube 5  $\mu$ l of sample was added to a total 20  $\mu$ l reaction volume with the addition of N2 primers to 250 nM and N2 probe to 125 nM. Samples were amplified using a Corbett Research Rotor Gene 6000. The protocol was by 95 C for 2 mins and 40 cycles of 95 C 5 sec, 60C 20 sec with read on the FAM channel.

Report Prepared by:		Approved By:	
Name:	Dr Robin Maytum	Name:	Michal Kisiel
Job Title:	Consultant Scientist	Job Title:	Quality Manager
Signature:	The Masters	Signature:	Mirl
Date:	22 <sup>nd</sup> November 2022	Date:	22 Nov 2022

# CliniSciences Group

#### Austria

Company: CliniSciences GmbH Address: Sternwartestrasse 76, A-1180

Wien - Austria Telephone: +43 720 115 580 Fax: +43 720 115 577

Email: oesterreich@clinisciences.com Web: https://www.clinisciences.com

#### Finland

Company: CliniSciences ApS Address: Oesterbrogade 226, st. 1, Copenhagen, 2100 - Denmark Telephone: +45 89 888 349 Fax: +45 89 884 064

Email: suomi@clinisciences.com

## Web: https://www.clinisciences.com

#### Iceland

Company: CliniSciences ApS Address: Oesterbrogade 226, st. 1, Copenhagen, 2100 - Denmark Telephone: +45 89 888 349 Fax: +45 89 884 064

Email: island@clinisciences.com Web: https://www.clinisciences.com

#### Netherlands

Company: CliniSciences B.V. Address: Kraijenhoffstraat 137A 1018RG Amsterdam, Netherlands Telephone: +31 85 2082 351 Fax: +31 85 2082 353

Email: nederland@clinisciences.com Web: https://www.clinisciences.com

#### **Portugal**

Company: Quimigen Unipessoal LDA Address: Rua Almada Negreiros, Lote 5, Loja 14, 2615-275 Alverca Do Ribateio - Portugal

Fax: +351 30 8808 052 Email: info@quimigen.com Web: https://www.quimigen.pt

Telephone: +351 30 8808 050

#### Switzerland

Company: CliniSciences Limited Address: Marktgasse 18 8302 Kloten -

Switzerland

Telephone: +41 (044) 805 76 81 Fax: +41 (044) 805 76 75

Email: switzerland@clinisciences.com Web: https://www.clinisciences.com

#### Belgium

Company: CliniSciences S.R.L Address: Avenue Stalingrad 52, 1000

Brussels - Belgium Telephone: +32 2 31 50 800 Fax: +32 2 31 50 801

Email: belgium@clinisciences.com Web: https://www.clinisciences.com

#### France

Company: CliniSciences S.A.S Address: 74 Rue des Suisses, 92000

Nanterre- France

Telephone: +33 9 77 40 09 09 Fax: +33 9 77 40 10 11 Email: info@clinisciences.com Web: https://www.clinisciences.com

#### Ireland

Company: CliniSciences Limited Address: Ground Floor, 71 lower Baggot street

Dublin D02 P593 - Ireland Telephone: +353 1 6971 146 Fax: +353 1 6971 147

Email: ireland@clinisciences.com Web: https://www.clinisciences.com

## Norway



Email: norge@clinisciences.com Web: https://www.clinisciences.com



### Spain

Company: CliniSciences Lab Solutions Address: C/ Hermanos del Moral 13 (Bajo E), 28019, Madrid - Spain Telephone: +34 91 269 40 65 Fax: +34 91 269 40 74

Email: espana@clinisciences.com Web: https://www.clinisciences.com



Company: CliniSciences Limited Address: 11 Progress Business center, Whittle

Telephone: +44 (0)1753 866 511 or +44 (0) 330 684 0982 Fax: +44 (0)1753 208 899 Email: uk@clinisciences.com

#### Denmark

Company: CliniSciences ApS Address: Oesterbrogade 226, st. 1, Copenhagen, 2100 - Denmark Telephone: +45 89 888 349 Fax: +45 89 884 064

Email: danmark@clinisciences.com Web: https://www.clinisciences.com

#### Germany

Company: Biotrend Chemikalien GmbH Address: Wilhelm-Mauser-Str. 41-43,

50827 Köln - Germany Telephone: +49 221 9498 320 Fax: +49 221 9498 325 Email: info@biotrend.com Web: https://www.biotrend.com



#### Italy

Company: CliniSciences S.r.I Address: Via Maremmana inferiore 378 Roma 00012 Guidonia Montecelio - Italy

Telephone: +39 06 94 80 56 71 Fax: +39 06 94 80 00 21 Email: italia@clinisciences.com Web: https://www.clinisciences.com



#### Poland

Company: CliniSciences sp.Z.o.o. Address: ul. Rotmistrza Witolda Pileckiego 67 lok. 200 - 02-781 Warszawa -Poland

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
Telephone: +48 22 307 0535 Fax: +48 22 307 0532

Email: polska@clinisciences.com Web: https://www.clinisciences.com



Company: CliniSciences ApS Address: Oesterbrogade 226, st. 1, Copenhagen, 2100 - Denmark Telephone: +45 89 888 349 Fax: +45 89 884 064

Email: sverige@clinisciences.com Web: https://www.clinisciences.com



Parkway, SL1 6DQ Slough- United Kingdom

IWeb: https://www.clinisciences.com

Company: Biotrend Chemicals LLC Address: c/o Carr Riggs Ingram, 500 Grand Boulevard, Suite 210 Miramar

Beach, FL 32550- USA Telephone: +1 850 650 7790 Fax: +1 850 650 4383

Email: info@biotrend-usa.com Web: https://www.biotrend-usa.com







